

Meet



Dame Condoms

Welcome to **Dame**,

We create products for pleasure, helping you connect with yourself, improve sexual wellness, and support overall wellbeing. →

# Ultra-Thin Condoms

Meet Dame Condoms: protection so thin, it feels like it's hardly there. Ultra-thin, body-safe, and perfect for those intimate moments that deserve more feel, less fuss.



## Get to Know Dame Ultra-Thin Condoms

- 100% Natural rubber latex
- Vegan-friendly
- Easy-tear foil packaging
- Body-safe lubrication
- Odor-free



# Using Dame Condoms

Latex condoms are designed to help prevent pregnancy, HIV/AIDS, and other sexually transmitted infections (STIs). They work by creating a barrier that reduces the risk of transmission.

---

1. **Handle with Care:** Tear open the package carefully to avoid damaging the condom. Avoid using fingernails, teeth, or sharp objects. Use a new condom for each act of intercourse—reusing condoms increases the risk of breakage, pregnancy, and STIs.
2. **Prepare:** Remove the condom from its packaging and apply a small amount of lubricant inside the tip. Using additional lubricant during vaginal or anal sex can help reduce the risk of breakage.
3. **Pinch the Tip:** Before any sexual contact, place the condom on the head of the erect penis with the rolled side facing outward. If uncircumcised, pull the foreskin back first. Pinch the tip of the condom between your thumb and forefinger to leave space for semen and prevent trapped air.
4. **Unroll:** Unroll the condom down to the base of the penis using your other hand. If the condom doesn't

unroll easily, it might be on backward. Discard it and start over with a new condom.

5. **Apply Lubricant:** To minimize the risk of breakage, apply additional lubricant to the outside of the condom before intercourse.

6. **During Intercourse:** If the condom breaks or leaks during use, stop immediately and withdraw. In case of a leak or burst, seek medical assistance within 72 hours. Emergency contraception is available over the counter in the U.S. and Canada and is most effective when taken as soon as possible, ideally within 72 hours of unprotected sex.

7. **Hold On Tight:** After ejaculation, hold the base of the condom tightly while withdrawing while the penis is still erect. This prevents the condom from slipping off and spilling fluids.

8. **Dispose Properly:** Wrap the condom in tissue and discard it in the trash. **DO NOT FLUSH CONDOMS DOWN THE TOILET.**

---

The FDA has not evaluated these statements. This product is not intended to diagnose, treat, cure or prevent disease.

Consult with a health professional.

---

## Caution

- Be sure to put on a condom prior to any sexual contact.
- **DO NOT USE OIL-BASED LUBRICANTS**, such as those made with petroleum jelly, mineral oil, vegetable oil, or cold cream, as they may damage the condom. Only use a water-based or silicone lubricant.
- Do not reuse condoms.
- Store condoms in a cool, dry place and avoid exposure to direct sunlight.
- If the latex material is sticky, brittle or obviously damaged, do not use the condom.
- If the color of the latex is uneven or changed, do not use the condom.
- Do not use if condom wrapper package is damaged.
- This product contains natural rubber latex, which may cause allergic reactions including anaphylactic shock.

## Questions?

Email: [support@dameproducts.com](mailto:support@dameproducts.com)

©2024 Dame Products Inc. dba Dame Products.

Dame Products are trademarks or registered trademarks of Dame Products Inc. All rights reserved



[Snap this code](#)  
to explore tips,  
and all things  
sexual health.